

Millennium line

Advanced Patient Care

MODEL ST370



FUNCTIONALITY

VERSATILITY

DURABILITY

ECONOMY

SAVION
ADVANCED PATIENT CARE

Features - Benefits

- the hydraulic shower trolley is a major cost saver enabling the washing and bathing of the most severely disabled patients to be carried out by only one patient nurse or attendant, and is also ideally suited for burns treatments.
- twin-pedestal hydraulic height adjustment provides maximum stability in every situation and safe even for obese patients.
- height adjustment pedals are located on both sides of the trolley.
- separate pedals serve for lowering the trolley - only light pressure needs to be applied to lower the platform gently and safely to comfortable low position - from either side.
- the mattress tray can be tilted to one side for draining and cleaning - the tilting mechanism is securely locked below the tray and is weight compensated by pneumatic gas spring - so requires no physical strength from attendants to operate.
- 4 x 200 mm (8") castors, with central-locking brakes, and one steering castor for perfect mobility, with brake pedals located at each corner.
- the shower tray is waterproof and gently padded, and is supplied with a removable head support pillow.
- the mattress interior is manufactured from chemical cross-link 20 mm thickness PA 300 EVA covered with heat sealed PVC coated polyester - the foam repels water and cannot become waterlogged even in the event of the cover splitting.
- quick-release weight compensated drop-down side panels enable patients to be moved in and out of the trolley safely and quickly.
- the sturdy ABS undertrolley housing - to protect the mechanism from damage and rusting - includes built-in storage space for patients effects as well as an oxygen cylinder cradle, enabling the trolley to be used for bathing patients requiring oxygen assistance.
- height adjustment range 64 - 95 cm

Construction

Twin-pedestal construction - the safest and surest way to ensure that the patient surface is always rigid and stable in every medical situation. The base frame, manufactured from durable and robust galvanized steel tubes, is covered by a sturdy and elegant moulded ABS cover - which also serves as handy storage rack if necessary.

The mattress platform is constructed from high-grade fiberglass sheet reinforced with steel bars for absolute safety, and the entire chassis is finished in epoxy-polyester powder coating - in a wide choice of colour finishes.

Mobility

The Millennium ST370 shower trolley is fitted with 8" (200 mm) castors, with 4-wheel locking and built-in steering mechanism which can be operated from any of the 4 corners of the stretcher by clearly indicated rocker pedals. The castors have thermoplastic rubber treads - ensuring longer life and smooth-running as well as improved mechanical behavior such as wear resistance, resistance to temperature extremes and improved mobility.

Colour Options

The standard colour is blue (RAL 5010); other colours are available on request.

Millennium line

MODEL ST370

Savion's Millennium Line new generation of patient care equipment - including hospital stretchers, ICU beds and shower trolleys - is designed for maximum patient comfort and safety - with minimum attendant effort and exertion



Standard features




*Central locking 8" castors
brake/steer pedals at
each corner
Drainage outlet and flexible
drainage hose.*



*Fold-under side rail
release lever.*



*Separate clearly marked
pedals for raising and
lowering located on both
sides of the trolley. Oxygen
cylinder storage cradle.*

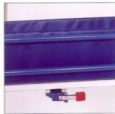

*Safety lock release lever for
pneumatic drainage and
cleaning tilting function.*

Specifications - Shower trolley model ST 370

Overall

length:	202 cm	80"
overall width:	74 cm	29"

Internal:

length:	180 cm	71"
width:	60 cm	23.5"

Height Adjustment Range:

minimum height with 8" castors	64 cm	25"
maximum height with 8" castors	95 cm	37.5"

Castors:

8" with central locking
one tracking
4 brake/steer pedals

Options

- The shower tray can be placed in absolutely horizontal position for bathing treatment for burns cases by adjustment pin located below the mattress surface.
- All stainless steel castors.
- Stainless steel side rails.



*Savion reserves the right to change specifications without notice